
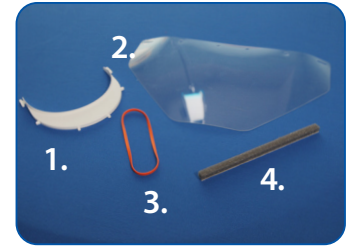
# Instructions for Face Shield Assembly (FSU1610HB3)



## Empty kit components from the bag:

1. Head gear
2. Shield
3. Latex free rubber band
4. Foam strip

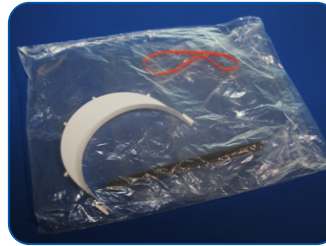
## Foam application (optional)

1. Remove paper from foam strip
2. Fold foam in half, adhesive side out
3. Place fold on the middle of the inside of the head gear
4. Apply foam to headgear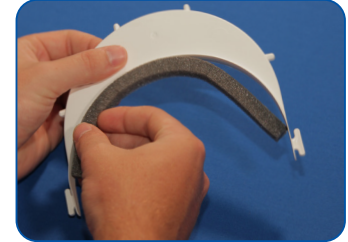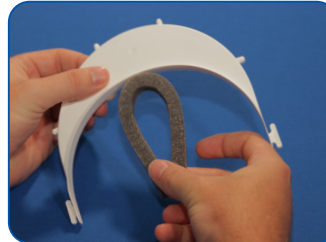



## Shield application

1. Starting at one side, attached shield to the T
2. Fold around and apply shield to each of the 4 notches
3. Pull over the T on the opposite side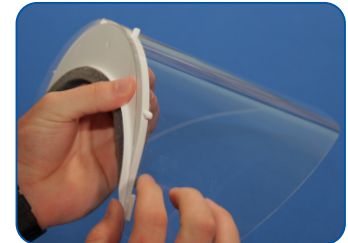

## Head fastener (latex free rubber band)

1. Attach the rubber band to each T
2. Place on head both bands in the back
3. If the shield is too tight, move one band to the top of the head

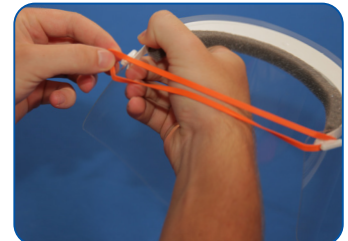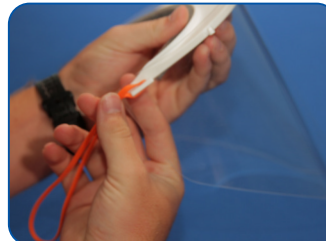## CAUTION

- This product is **NOT** intended for use in presence of high intensity light source or flammable gas.
- This product is **NOT** sterilized.
- Use of this face shield alone will not prevent infection from microbes or viruses.

## CLEANING AND DISINFECTING INSTRUCTIONS

- To clean and disinfect, use a Health Canada authorized hard-surface disinfectant. Refer to Health Canada's authorized disinfectants that may be used against SARS-CoV-2, available on the Health Canada website. The recommended types of disinfectants for this face shield and headgear are those based on quaternary ammonium compounds or sodium hypochlorite.
- Use the disinfecting product according to the manufacturer's application instructions and be sure to follow the exposure time requirements. Consider the face shield and headgear you are about to clean contaminated at every surface and take appropriate measures, including wearing gloves and additional protective equipment.
  1. Disassemble the faceshield and headgear. Remove and discard the optional foam strip if applied.
  2. Carefully apply the disinfectant product to the entire surface of the face shield and headgear, both inside and out.
  3. Reapply as necessary to ensure the entire surface remains wet for the exposure time required by the disinfecting product.
  4. Following treatment, dry the surface with clean absorbent towels.
  5. Product residue may be removed with water and absorbent towels.
  6. After treatment, remove gloves and perform hand hygiene.
- Most disinfecting formulations will not affect the material, but discoloration or haziness indicates incompatibility between the face shield material and the disinfecting formulation. Face shields or headgear that show visible effects after treatment should not be used.

For more information contact us at [ppeproducts@novolex.com](mailto:ppeproducts@novolex.com)

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